


MedXL Needlefree Injection Site Valves

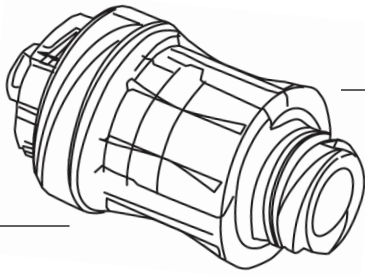

Instructions for Clinical Use

Accessing the Swabable Valve


STERILE R



CONNECTION

1. Using aseptic technique, remove the sterile device from the package.
Do not use if packaging is not intact.
2. Using a sterile alcohol wipe, swab the surface of the valve (FIG. 1). Let it air dry.
3. Carefully connect the syringe or extension set to the valve by **pushing the syringe or other Luer connection straight into the swabable valve using a clockwise, twisting motion.**

DO NOT try to insert at an angle or try to pry open the slit in the valve.

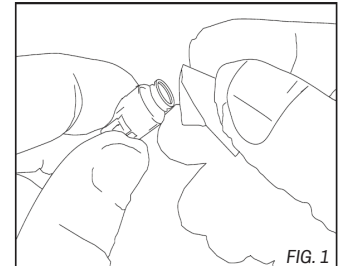

Note: When using rotating collar MLL connectors ensure that collar is rotated and connection is secure.

Male Luer Lock (MLL)

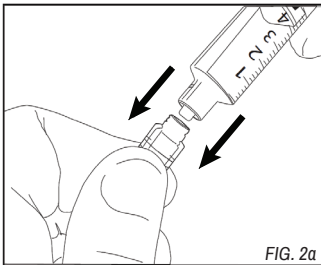


FIG. 2a

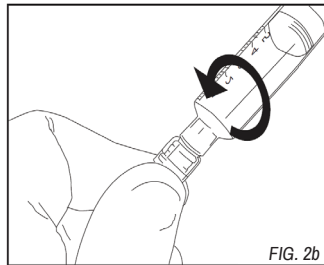


FIG. 2b

Male Luer Slip (MLS)

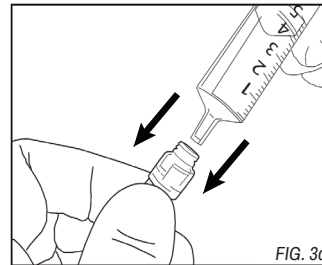


FIG. 3a

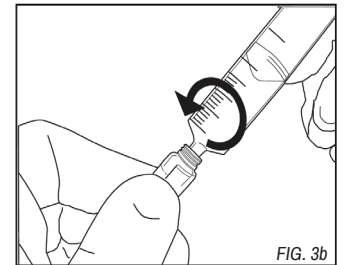


FIG. 3b

DISCONNECTION

4. Twist counterclockwise to disconnect. The swabable valve completely closes after each use and therefore does not require a separate injection site cap.
5. Flush the swabable valve device after each use per facility protocol.
6. Change swabable device per facility protocol.
7. Dispose of used device per facility protocol for biocontaminated materials.

The MedXL valve is used in several different medical devices.
The technique for accessing the valve is the same.

CAUTION

- To prevent valve damage, **DO NOT USE** needles or blunt cannula to access the swabable valve.
- Carefully follow the directions to maintain the valve integrity.
- Only use standard Luer connection devices; non-standard syringes or connectors can damage the swabable valve. A standard luer connection must conform to the harmonized standards, ISO 80369-7, ISO 594-1 and/or ISO 594-2. Syringes and male luer connectors have a large variety of configurations and can vary significantly in design and dimensions.
- **DO NOT OVER-TIGHTEN** connections.
- **DO NOT USE** any instruments to tighten connections.