


# Praxiject™ Saline

0.9% NaCl Prefilled Syringes

## WARNINGS AND PRECAUTIONS :


- Use aseptic technique.
- Single use only. Do not reuse.
- For IV flush only.
- Do not use if package is opened or damaged.
- Do not use if the cap/adaptor on the syringe is not intact.
- Do not resterilise.
- Sterile unless package opened or damaged.

## Usage Guidelines

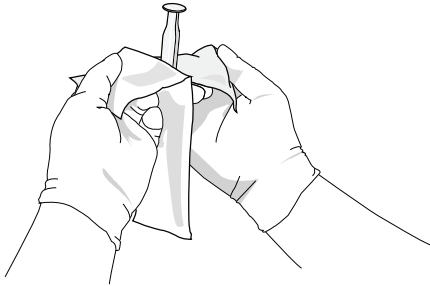


**1**

Aseptically open package containing Praxiject™ Saline prefilled syringe\*. Visually inspect the contents of each prefilled syringe for clarity, particulate matter, precipitate, discoloration and leakage prior to use. Do not use if any of the aforementioned is observed.

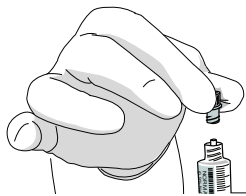


**2**

Remove and discard syringe cap.

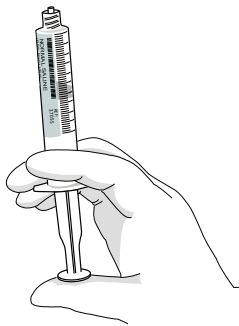

**3**

Expel air from syringe. Do not allow air to be trapped in fluid path. Use only with compatible luer lock connectors.

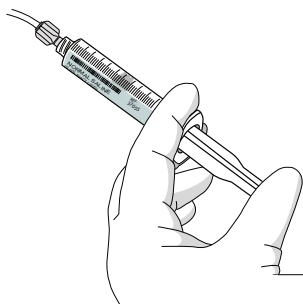

**4**

Following the institution's policy, instill the required volume of Praxiject™ solution.



**5**

Remove and discard unused portions and empty syringes according to the institution's biohazardous waste policy.

\* Praxiject™ products are sterile field compatible