

MedXL Vial Adapters



PRAXIJECT™ vial adapters are the ideal solution for quick and safe transfer of liquids and reconstituted powders from vials.

- PRAXIJECT™ vial adapters are easy to use, fewer steps and fewer sharps.
- PRAXIJECT™ vial adapters enable safe and rapid drug transfer and reconstitution.
- PRAXIJECT™ vial adapters are needleless. Exposure to needle stick injuries are reduced.
- PRAXIJECT™ vial adapters are compatible with all standard luer lock and luer slip syringes.
- PRAXIJECT™ vial adapters are available with a swabable valve that maintains sterility and eliminates stopper integrity issues associated with multidose rubber stoppers.

Instructions for use

Use aseptic technique. Wash hands with soap and hot water before use.


Step 1

Remove the cap + disinfect the injection site on the drug vial.




Step 2

Remove the cover from the vial adapter package.

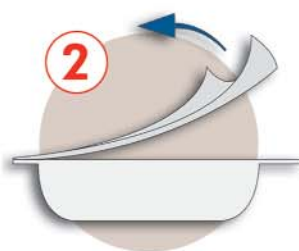

Step 3

Remove and discard the spike cover.

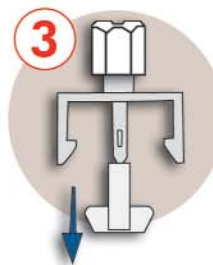

Step 4

Push the adapter on the vial injection site until it clicks in place.



The vial is now ready for access using an empty syringe for withdrawing liquids or with a diluent prefilled syringe for reconstituting powders.

Catalog

Description

Packaging

3022	Large vial adapter with Female L/L + Cap, sterile.	40/bx	160/cs
3023	Small vial adapter with Female L/L + Cap, sterile.	40/bx	160/cs
3025	Large vial adapter with swabable valve, sterile.	40/bx	160/cs
3026	Small vial adapter with swabable valve, sterile.	40/bx	160/cs

Distributed by